

# Hip Belt Extension: Universal

The ISO Preferred Lumbar Belt Extension: Universal provides an extension to the hip belt for waist sizes over 56 inches.



ISO Preferred Hip Belt Extension: Universal (HI204) Applies to two different types of Hip Braces, the ISO Preferred Post OP Hip Brace & The Clam Shell models.

**CLAM SHELL MODELS:** - Belt attaches to brace with clamshell gripping material:

ISO PREFERRED  
DR MEDICAL

ISO-HI206  
HB-166D

**BEFORE APPLICATION:** You will be applying 2 Belt Extensions, one on each side of the back brace. Detach and extend one side of the hip brace belt at a time.

**WHEN COMPLETE, MAKE SURE THE FRONT SIDE WITH THE HOOK AND LOOP FASTENING IS ON THE RIGHT SIDE OF THE PATIENT'S BODY WHEN WORN.**

- 1. APPLY THE EXTENSION:** On one side, open up the clam shell fastener, detach the hip belt and apply Extension Belt, facing down.
- Attach the original hip belt to the Belt Extension by positioning leftward or rightward to add the desired number of inches for this side. Secure original belt to the Extension. Apply Belt Extension to the other side of the brace making sure both sides are even. Apply brace as instructed on product instructions.

1



2





## INTRODUCTION

Your **ISO Preferred Hip Belt Extension: Universal (HI204)** is designed to provide your body with the support and stability that it needs to successfully manage or eliminate your mobility or pain concerns. It is critical that you follow your orthopedic professional's directions closely to ensure that the apparatus is worn properly and in the manner that will be most beneficial in addressing your issues. For correct application, please read the enclosed instructions and watch the on-line instructional video (if available) at [www.ISOpreferred.com](http://www.ISOpreferred.com) so that this device can provide you with superior long-term usability and care.

## USAGE RECOMMENDATIONS

It is vital that you read the included instructions and when applicable watch the instructional video for the **ISO Preferred Hip Belt Extension: Universal (HI204)**. The brace is intended for application by qualified professionals only. It is not recommended for patients to fit or apply or adjust their new brace without an orthopedic professional's expertise, unless otherwise instructed. Further questions regarding adjustments or changes, frequency of use, and/or duration of use should be presented to your orthopedic professional, as they are familiar with the specific needs and conditions, for which this device was prescribed.

## CARE AND CLEANING INSTRUCTIONS

Please do not place your brace in the washing machine or dryer at any time. When preparing to clean your **ISO Preferred Hip Belt Extension: Universal (HI204)** you will want to separate the parts where applicable and care for them individually.

- Do not dry clean, iron, bleach, or use an abrasive cleaning instrument on any part of the brace. Failure to follow these instructions may damage your brace and void the manufacturer's warranty and it may no longer work properly.
- Hand wash each section in cold water with a mild detergent. Lay flat to air dry. Allow all parts to air dry prior to reassembling the device.

## CAUTION

The **ISO Preferred Hip Belt Extension: Universal (HI204)** is for single patient use only. While every effort has been made to obtain maximum strength, function and durability, there is no guarantee that injury will be prevented through the use of this device, especially if the instructions are not followed correctly.

**DO NOT OVERTIGHTEN.**