



### The Comfortmax 360 Universal Hinged Knee Brace

- Excellent choice for patients who are unable to pull a knee sleeve over their foot
- Medial and lateral hinges provide support for the wearer
- Unique wrap around design provides wearer with an exact comfortable fit



#### HOW TO APPLY:

Unfasten all straps. Position the back of the brace with the hollow back of the knee so its centered behind the knee. Align the hinges with the center of the knee joint. Wrap around the front and secure with the Velcro straps. Support should fit snug, but not so tight as to restrict circulation. Hinges should be aligned with the prominent bones on each side of the knee. Secure the straps beginning with the top strap and then the bottom strap.

#### CARE INSTRUCTIONS:

Fasten closures. Hand wash in cold water with mild soap. Wrap in a towel to remove excess water ( do not twist or wring ). Air dry away from heat. Do not tumble dry. Do not bleach.

#### CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

#### WARNING:

Contact neoprene material with skin may cause irritation. Persons with known susceptibility to dermatitis or with broken skin should avoid contact with neoprene. Anyone who develops skin irritation from contact with this material should discontinue its immediately. Should not be worn by persons with phlebitis or circulatory disorders. Not recommended to be worn while sleeping.

Item #: KB – 159 (Universal - one size fits most)

SUGG. HCPCS CODE: L1820 / L1810 / L1812

DR Medical  
203 Rio Circle Site. B  
Decatur, GA 30030  
[www.drmedsupplies.com](http://www.drmedsupplies.com)

尺寸：21x14.8cm